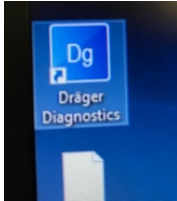



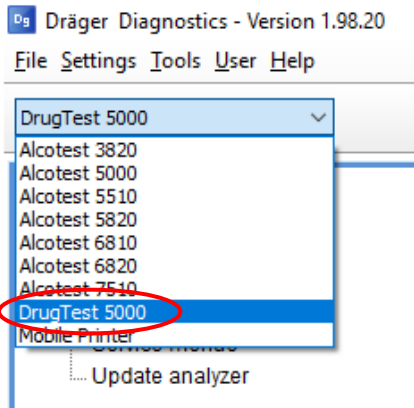
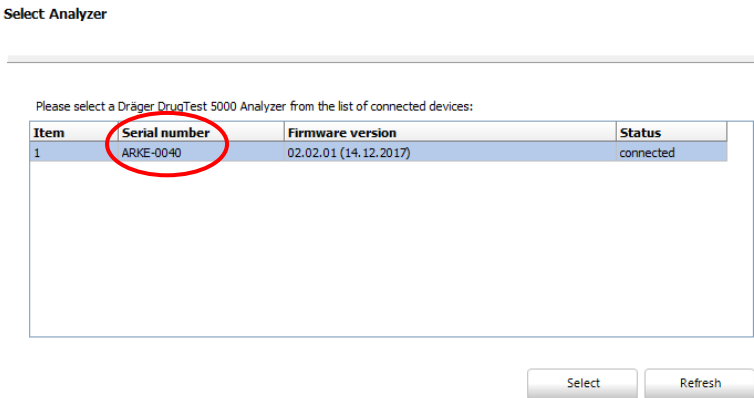
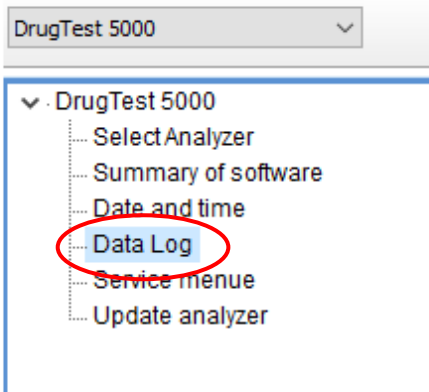
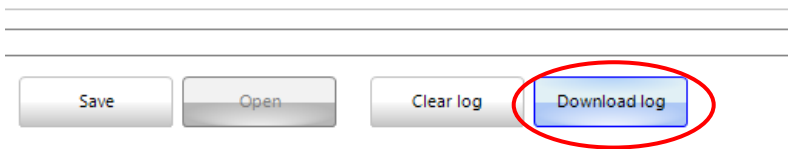

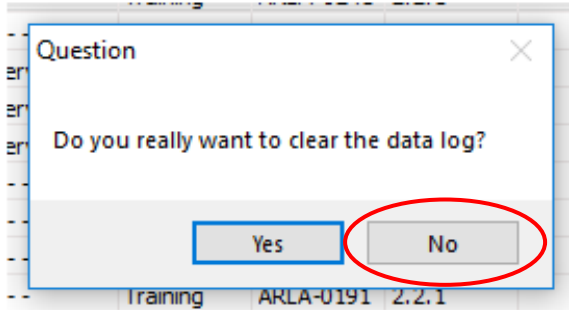
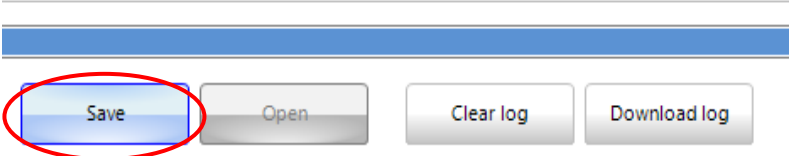


Downloading Results from the DrugTest® 5000

Download results stored on your DrugTest5000 device to your PC in an Excel file.

1	Ensure the Dräger Diagnostics software is downloaded on your PC (see additional installation instructions). Launch the program.	
2	Click the "log in" link in the upper right corner.	
3	Log in with the following credentials: Username: Operator Password: Ops@5000!	
4	Connect your DT5000 to the PC using the provided USB cable. If you cannot locate the original cable, the DT5000 accepts normal USB printer cables as pictured.	

5	Using the product dropdown menu in the upper left, select DrugTest5000.	
6	In the left-hand menu, choose "select analyzer" and ensure the serial number of your device is listed (click "refresh" if it is not), then click "select".	
7	In the left-hand menu, choose "Data Log".	
8	Click "download log". You will see a preview of the device's data on the screen.	

<p>9</p> <p></p>	<p>Next, you will be prompted to clear the data off the analyzer.</p> <p>It is recommended that you select "No" at this point, to first ensure you have a usable exported file (the data cannot be retrieved once cleared).</p> <p>You can return to this menu and clear the data log at a later time.</p>	
<p>10</p>	<p>Click "Save".</p>	
<p>11</p>	<p>Select where you would like to save the file on your PC and click "save."</p> <p>You have now successfully downloaded the data from your DrugTest 5000.</p> <p>Ensure the Excel file opens properly. At this point, you can go back to clear the log from the device, if desired. The device stores up to 500 tests. Once the data is deleted from the analyzer, it cannot be retrieved.</p>	