



## Dräger Training Cassette Instructions

1	<p><b>NO SAMPLE COLLECTION IS REQUIRED TO USE THE TRAINING CASSETTE. DO NOT EXPOSE TO ORAL FLUID.</b></p> <p>The yellow cassette/yellow cap will simulate a <b>negative</b> result.</p> <p>The yellow cassette/red cap will simulate a <b>non-negative</b> result.</p> <p>Instructions are the same for both types.</p>	
2	<p>Remove the clear cap and insert the cassette and plunger cap into the machine as directed. Close the door.</p>	

3 Enter the donor info with the keyboard if desired, or press "save" to skip this step and run the test.

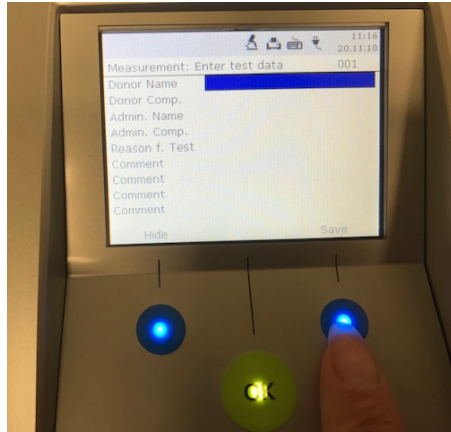



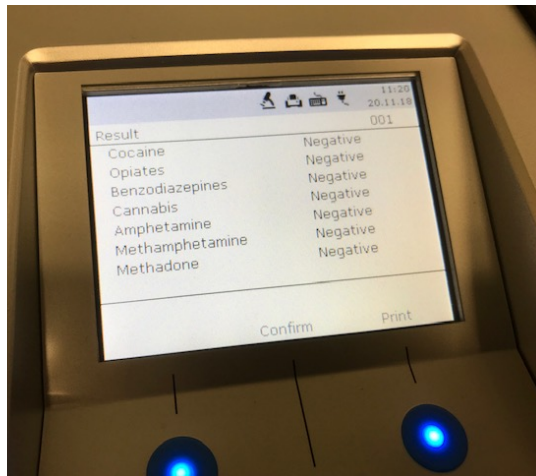4 The test will run for approximately 4 minutes and display the results on the screen.  
  
Press "confirm" to continue or "print" to print the results to a connected Dräger mobile printer.



5 Remove the cassette from the machine and close the door.



6	<p><b>IMPORTANT</b> - Reset the cartridge by grasping the collection sponge and pulling it back to the fully upright position.</p>	
7	<p>From the bottom of the cassette, slide the black test panel back to the front of the housing using a pen or other small object in the slot.</p>	
8	<p><b>NO NOT DISCARD</b> the training cassette. It is now ready to re-use.</p>	